Flamigel®

Indicaties

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Flamigel®

Indicaties

Flamigel® is a hydro-active colloid gel for the treatment of minor wounds which supports the fast healing of skin by covering the wound and creating an optimal moist healing environment. Flamigel® is for use on superficial burns and minor wounds (e.g. grazes, cuts). Flamigel® may also be used in open wounds (without substantial breach of the dermis) and superficial bedsores (only under supervision of a doctor or a nurse).

Properties

Flamigel® hydro-active colloid formulation can quickly help soothe the pain of minor wounds due to its cooling effect. It helps to create optimal healing conditions to accelerate cell renewal, allowing the wound to heal fast and so reduce the likelihood of scarring. The gel also cushions the wound to provide a barrier against external contamination. Suitable for the whole family this gel is easily applied, non-sticky and non-greasy.

Flamigel® can be used for dry or wet wounds:

- In case of a dry wound Flamigel® turns the dry wound into a moist one ("hydrogel" effect). The dry wound absorbs the water contained in Flamigel® and promotes the dissolution of necrotic/damaged tissue.
- In case of a wet wound, the hydrocolloid of Flamigel® is activated and consequently Flamigel® absorbs excessive fluid from the wound and restores the water balance in the wound ("hydrocolloid" effect).

How to use

Do not use on extensive weeping or infected wound unless recommended by your doctor. Wash hands before and after treating the wound. Clean and dry the wound gently before applying the gel.


For superficial burns and minor wounds (e.g. grazes, cuts)

Carefully apply Flamigel® to the wound (without rubbing) once or twice per day (in case of sunburn apply twice to six times daily). If necessary, cover the wound with a plaster or dressing and change daily.

For open wounds (without substantial breach of the dermis) and superficial bedsores (only under supervision of a doctor or a nurse):

Apply a thick layer of Flamigel® (5 mm) onto the wound minimising contact with the wound edges. Cover Flamigel® with a sterile non-adhesive dressing and fix with gauze or a bandage. Leave Flamigel® in place as long as the gel structure is intact. When new skin cells start to form and the wound is closing, only apply a thin layer of Flamigel® to prevent weakening. If necessary, protect the wound edges.

- Open wounds: change daily.
- Superficial bedsores: change daily or every two days.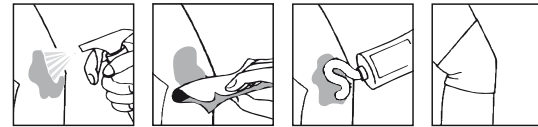

clean dry apply cover

Removing the gel:

- Wash hands before and after removing Flamigel®.
- Flamigel® can be removed simply and painlessly with wet gauze.

Precautions

- For external use.
- Side-effects such as irritations or allergic reactions are rare.
- Do not use Flamigel® when there is a known allergic reaction to parabens or any of the ingredients.
- Do not use Flamigel® on the eyelids or in the eyes. If Flamigel® comes into contact with the eyes rinse the eyes with running water and consult a doctor.
- On extensive weeping or infected wounds, consult a doctor before using Flamigel® because other complementary treatments might be necessary.
- If the wound becomes swollen, reddened or painful, or doesn't start to heal within a few days seek medical advice.
- If you are pregnant or breast-feeding, ask a doctor before using Flamigel®.

Storage conditions

- Store at room temperature (< 25°C), in a dry place and in the original packaging.
- Close the tube after use.
- Do not use Flamigel® after the expiry date shown on the pack.

Ingredients

Acid colloïdal hydrocolloïd, Arginine, Purified water, Macrogol, Branch Chained Fatty Acid (BCFA), Methyl-p-hydroxybenzoate (E218), Propyl-p-hydroxybenzoate (E216), Disodium EDTA.

How supplied

Tubes of 10 g, 50 g and 250 g. Not all pack sizes may be marketed.

Manufacturer

 Flen Pharma NV/SA, Blauwesteenstraat 87, B-2550 Kontich, Belgium
www.flenpharma.com – info@flenpharma.com MADE IN BELGIUM

Distributor

 NOVARTIS Novartis Consumer Health NV/SA, Medialaan 30/5 - 1800 Vilvoorde (Belgium)

Medical Device