## Flaminal® Forte

### Eigenschappen

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## Flaminal® Forte

### Proprieties

Flaminal® Forte is an AlginoGel® with alginates and an antimicrobial enzyme system. Flaminal® Forte supports the healing of a wound in a number of ways:

- Flaminal® Forte keeps the wound clean
- Flaminal® Forte keeps the wound moist
- Flaminal® Forte offers antimicrobial protection
- Flaminal® Forte is safe for the skin and wound tissue
- Flaminal® Forte reduces wound odour
- Flaminal® Forte reduces excessive protease activity (protease = enzyme that breaks down proteins)
- Flaminal® Forte protects against bacteria in a biofilm and reduces bio-film mass
- Flaminal® Forte protects the wound border

These are the prime properties for ensuring rapid wound healing.

Flaminal® Forte is indicated for moderately to heavily exuding wounds such as:

- heavily exuding 2nd degree burns (deep, superficial)
- heavily exuding ulcers
- exuding pressure sores
- oncology wounds
- pre-operative wound-bed preparation

### How to use

Applying or re-applying Flaminal® Forte to a wound is easy and painless. Proceed as follows:

- First clean and rinse the wound (e.g. with Flamirins®)
- Dab and dry carefully
- Cover the entire wound-bed with Flaminal® Forte
- Cover Flaminal® Forte with a dressing. The choice of dressing should be based on the type of lesion:
  - Moderately exuding wound: transparent film (polyurethane) or paraffin gauze with a non-adhesive bandage
  - Heavily exuding: non-woven gauze with a non-adhesive bandage or with a hypoallergenic, adhesive tape
  - Very heavily exuding: absorbent non-adherent compress fixed by a non-adhesive bandage or a hypoallergenic, adhesive tape

### Good to know

- Check the dressing daily. If it is leaking or if there is insufficient gel, Flaminal® Forte should be changed and a new dressing applied.
- During Flaminal® Forte treatment, dry whitish alginate flakes may appear on the wound border. They should not be removed. The flakes will prevent wound border maceration. Maceration slows the healing process of the wound.
- When dry flakes or alginates appear in the wound, the wound is too dry for Flaminal® Forte. Use Flaminal® Hydro.
- The dressing with Flaminal® Forte can remain in place as long as the gel structure is intact (1 to 4 days, depending on the amount of exudate).
- In the initial phase of the treatment the wound may seem to become larger. This is a normal part of the healing process in which dead tissue is eliminated.
- The colour of the alginates may change slightly due to their natural origin. Colour changes have no effect on the quality of Flaminal® Forte.

### Special wounds

- Leg ulcer: Adhesives often cause allergic reactions, particularly in patients who suffer from leg ulcers. Cover with a non-adhesive bandage.
- Undermined wounds: Place sterile gauze tampons in the cavities as a support; apply a sufficient amount of Flaminal® Forte in the cavities between the tampons.
- Fistulas and small deep cavities: Inject Flaminal® Forte carefully into the cavity using a dedicated cannula.
- Heavily exuding wounds: Apply a thicker layer of Flaminal® Forte.
- Slightly exuding wounds: Apply Flaminal® Hydro.
- Exuding heel ulcers: Apply Flaminal® Forte and cover with a V-shaped gauze compress.

### Precautions

Some wounds may indicate a complementary treatment. Consult a physician or wound specialist.

Flaminal® Forte can be used on infected wounds under medical supervision. Alginates have a faint odour. This odour has no influence on the healing process. Should there be a sudden distinct odour, consult a physician or wound specialist. Flaminal® Forte is not indicated in patients with a known allergy to one of the components (see under "Compositie").

Flaminal® Forte cannot be used on the eyelids or in the eye. Should it come into contact with an eye, rinse the eye thoroughly with running water and consult a physician.

### Storage

Keep at room temperature (below 25°C) in a dry place in the original pack. The expiry date (USE BY) Year/Month is the first day of the indicated month and year.

Recap the tube or jar carefully immediately after use.

Once opened, and if recapped carefully, a tube of Flaminal® Forte can be stored and used until the expiry date on the tube and box. A jar of Flaminal® Forte 500g however cannot be used more than one week after it is first opened.

We recommend that a pack of Flaminal® Forte be used for one patient only (single patient).

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